



Collect your CHLabs kit and paperwork.

Ensure the box is sealed.

Open the box and check you have the following

1 pottle with blue sealed lid

3 tubes and 3 sample security seals

Page of barcode labels

Specimen bag with red tape still attached and 1 shipping seal.



Remove blue lid, fill the pottle with urine, replace lid, then remove the yellow sticker. Push the tube onto the needle as shown. Fill all three tubes.

Write the donors full name on each tube and attach a barcode. **Both** identifiers are required.

Sign and date the security seals. Place a security seal over the end of each tube.



Complete the documentation, barcode must be attached:
-if you have performed on-site screening tick the "MS confirmation" only box and the drug class/classes that were non-negative on your screen.
-if you have not performed on-site screening, tick the "Screen &MS Confirm" box.



Place the tubes and paperwork into the tamperproof bag, then into the box and attach the signed and dated shipping seal where the box seal had been. Place the box in the yellow courier bag provided by CHLabs and return to CHLabs.

