


**Orders: Internal via Oracle - Tube #265645 | Straw #267967**  
**For external users - order direct via Becton Dickenson**

1. Patient voids urine into collection boat or pottle
2. Use the transfer straw to draw urine into tube
3. Discard straw into a sharps container
4. Label tube

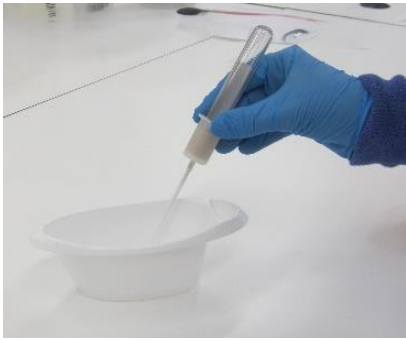

**Microbiology urine requests\* - collect 1 x 10ml tube but half fill to 5ml.**

**Other non-microbiology tests\* - collect 1 x 10ml tube and fill to the top of the tube label.**

\* collect two tubes if microbiology and biochemistry tests are required (see also exceptions below).

**STI PCR requests – do not use these tubes. Use a pottle and a multi-collect orange top tube.**

### **Important Exceptions**

**Samples for the tests below cannot be sent in the beige top tube due to volume or other requirements. They will be rejected for testing.**

**Cytology**

**Genetics**

**Leptospira**

**Mucopolysaccharides**

**Organic Acids**

**TB Urines**

**Urine Protein Electrophoresis (Bence Jones Protein)**

Refer to the laboratory test guide for specimen requirements for the tests above: [www.chl.co.nz/test-manager/](http://www.chl.co.nz/test-manager/).

**No change to 24hr urine collection requirements**